

## Your SureMed MultiMed 31-Day Starter Kit



Your SureMed 31-day starter kit includes: 26 cards, 26 blisters, a filling foam and sealing roller.

## Tips


- This blister packaging is not child resistant, please store beyond the reach of children.
- Keep unused cards in a clean, dry and air conditioned location.
- If there is a slight tug while peeling off the pressure sensitive liner try peeling from a different corner.

## Our Medications



We'll be filling 2 cards, 1 card with five morning medications and 1 card with four evening medications.

## 1. Remove Liner

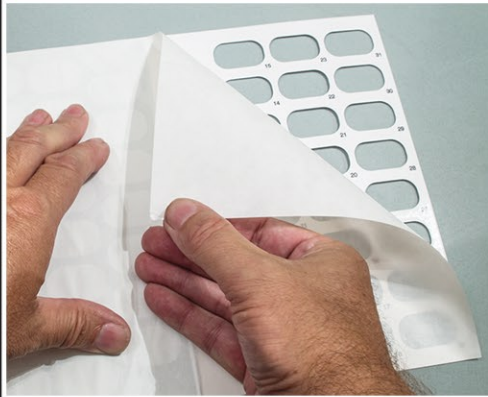

Start with the morning card, remove the pressure sensitive liner revealing the sealing surface.

## 2. Filling Foam



Place the card on the filling foam template (dry glue side up).

## 3. Insert Blisters



Insert the blister sheet into the card.

## 4. Filling Morning - Pill #1



You take 5 different medications in the morning. Place this first medication in each of the 31 blisters.

## Filling Morning - Pill #2



Place your second medication into each of the 31 blisters.